



## Scleroderma<sup>12</sup> IgG

Order Code: SCL12DIV-24

BlueDiver Protocol: 02

### 1. INTENDED USE

BlueDiver Dot Scleroderma<sup>12</sup> IgG is an Immunodot kit intended for the detection, in human sera only, of IgG autoantibodies against the antigens Scl-70, CENP-A, CENP-B, PM-Scl 100, PM-Scl 75, Ku, RNA Polymerase III, RNP 68kD/A/C, Th/To, Fibrillarin, NOR-90 and SSA/Ro 52kD.

This kit is intended to confirm results of patterns obtained by immunofluorescence, the screening and reference method in autoimmunity; the kit is intended as an aid in the diagnosis of several autoimmune diseases (for more details, see 11.5 Autoantibodies diagnostic values).

The test is intended for a large, routine population. This kit is strictly reserved for professional use in clinical analysis laboratories. This kit is strictly foreseen as an automated test and can only be used in a BlueDiver Instrument Model I or II (hereinafter named BDI I or BDI II respectively).

Detection of the different IgG autoantibodies can be either qualitative (see point 10.1) or semi-quantitative (see point 10.2).

For a semi-quantification of the test results, it is necessary to use the BlueScan scanner system and the Dr Dot software. This system is not included in the BDI I but is included in the BDI II (see point 4).

### 2. PRINCIPLE OF THE TEST

This kit and all its components are intended to be performed exclusively on the BDI I or II.

The test is based on the principle of an Enzyme Immunoassay. The strips are composed of a membrane fixed on a specific plastic support. During the automated test procedure, the BDI sequentially incubates the strips in the wells of ready-to-use reagent cartridges. Briefly: the strips are first incubated with diluted patients' sera. Human antibodies, if present, bind to the corresponding specific antigen(s) on the membrane. Unbound or excess antibodies are removed by washing. Upon further incubation into AP-conjugated goat antibodies against human IgG, the enzyme conjugate binds to the antigen-antibody complexes. After removal of excess conjugate by washing, the strips are finally incubated into a substrate solution. Enzyme activity, if present, leads to the development of purple dots on the membrane pads. The intensity of the coloration is directly proportional to the amount of antibody present in the sample.

The kit is composed of 24 single-use tests.

### 3. KIT CONTENTS

**Prior to any use of the kit, please check that all the items listed are present. Please also check if the characteristics of the product are corresponding to those described hereafter.**

**If one of the items is missing or damaged, please do not use the kit and contact your distributor.**

#### 3.1 COMPONENTS




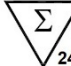
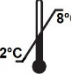









| Strips         | 3 x 8 units on plastic supports breakable individually sealed in aluminium pouch. Each strip is for single use.<br><br>14 dots on each:<br>1 positive control (RC)<br>12 antigens<br>1 negative control (CO) | STRIP  | CARTRIDGE  |
|----------------|--|--|--|
| Cartridge      | 24 units having each 7 compartments; sealed. Each cartridge is for single use  |  |  |
| Diluent buffer | I <sup>st</sup> position, 1 x 1,4 ml (yellow)<br><i>contains</i> H <sub>2</sub> O • TBS • NaCl • Tween • BSA • Preservatives • dye • antifoam emulsion   | RC —<br>Scl-70 —<br>CENP-A —<br>CENP-B —<br>PM-Scl 100 —<br>PM-Scl 75 —<br>Ku —<br>RNA Polymerase III —<br>RNP 68kD/A/C —<br>Th/To —<br>Fibrillarin —<br>NOR-90 —<br>SSA/Ro 52kD —<br>CO — | DIL 1,4 mL<br>II WASH 1,4 mL<br>III WASH 1,4 mL<br>IV WASH 1,4 mL<br>V CONJ IgG 1,4 mL<br>VI WASH 1,4 mL<br>VII SUB 1,4 mL |
| Wash buffer    | II <sup>nd</sup> , III <sup>rd</sup> , IV <sup>th</sup> and VI <sup>th</sup> position, 4 x 1,4 ml (colourless)<br><i>contains</i> H <sub>2</sub> O • TBS • NaCl • Tween • Preservatives • antifoam emulsion  |  |  |
| Conjugate      | V <sup>th</sup> position, 1 x 1,4 ml (red)<br><i>contains</i> H <sub>2</sub> O • TBS • NaCl • KCl • MgCl <sub>2</sub> • AP-conjugated goat anti-human IgG • Preservatives • dye • antifoam emulsion          |  |  |
| Substrate      | VII <sup>th</sup> position, 1 x 1,4 ml (pale yellow solution)<br><i>contains</i> H <sub>2</sub> O • Preservatives • MgCl <sub>2</sub> • TBS • NBT stabilizer • NBT • BCIP                                    |  |  |

Abbreviations in alphabetic order:

AP = Alkaline Phosphatase; BCIP = Bromo-Chloro-Indolyl-Phosphate; BSA = Bovine Serum Albumin; KCl = Potassium Chloride; MgCl<sub>2</sub> = Magnesium Chloride; NaCl = Sodium Chloride; NBT = NitroBlue Tetrazolium; TBS = Tris Buffer Saline

For more information on the composition and concentration of the active ingredients used, please refer to the MSDS available on request or on [www.d-tek.be](http://www.d-tek.be).

### Symbols used on kit labels

|   |                                    |   |                                      |
|---|------------------------------------|---|--------------------------------------|
|  | Consult instructions for use       |  | CE Mark + Notified Body              |
|  | In Vitro Diagnostic Medical Device |  | For 24 uses                          |
|  | To be stored from 2-8°C            |  | Reference                            |
|  | Batch Number                       |  | To be protected from direct sunlight |
|  | Use-by date                        |  | Manufacturer                         |
|  | Cartridge                          |  | Do not re-use                        |
|  | Strip                              |  | Caution                              |

### 3.2 Antigens used

|                    |   |
|--------------------|---|
| Scl-70             | DNA topoisomerase I (recombinant, human, expressed in Baculovirus-infected Sf9 cells)   |
| CENP-A             | Centromere Protein A (recombinant, human, expressed in Baculovirus-infected Sf9 cells)  |
| CENP-B             | Centromere Protein B (recombinant, human, expressed in Baculovirus-infected Sf9 cells)  |
| PM-Scl 100         | Polymyositis-Scleroderma antigen (100 kD subunit) (recombinant, human, expressed in Baculovirus-infected Sf9 cells)                         |
| PM-Scl 75          | Polymyositis-Scleroderma antigen (75 kD subunit) (recombinant, human, expressed in Baculovirus-infected Sf9 cells)                          |
| Ku                 | Regulatory subunit of DNA-dependent protein kinase (70/80 kD heterodimer) (recombinant, human, expressed in Baculovirus-infected Sf9 cells) |
| RNA Polymerase III | 155 kD subunit (RPC 155) of the RNA Polymerase III complex (Recombinant, human, expressed in E.coli)  |
| RNP 68kD/A/C       | Mixture of 68 kD, A and C proteins from snRNP particles (recombinant, human, expressed in Baculovirus-infected Sf9 cells)                   |
| Th/To              | RNase P nucleolar protein complex, Rpp 25 subunit (recombinant, human, expressed in E.coli)   |
| Fibrillarin        | Fibrillarin protein (recombinant, human, expressed in E.coli)   |
| NOR-90             | « Nucleolus-Organizing Regions » antigen, 90 kD (recombinant, human, expressed in E.coli)   |
| SSA/Ro 52kD        | E3-ubiquitin ligase (Tripartite motif protein 21, TRIM21) (recombinant, human, expressed in Baculovirus-infected Sf9 cells)                 |

### 3.3 Reactive ingredients

| Substance                                | Origin  | Intend purpose in Scleroderma kits                          | Concentration in Scleroderma kits                       | Purity  |
|--|---|---|---|---|
| Goat anti-human IgG-alkaline phosphatase | Animal (Goat)   | Secondary antibody (detection antibody) in conjugate buffer | < 0,1 µg/ml in conjugate buffer                         | Unknown.<br>No antibody is detectable to non-immunoglobulin serum components. |
| Scl-70 antigen                           | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips                        | <0,5 mg/ml<br>One Scl-70 spot = 0,5µl on each strip     | >80%  |
| CENP-A antigen                           | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips                        | <0,5 mg/ml<br>One CENP-A spot = 0,5µl on each strip     | >80%  |
| CENP-B antigen                           | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips                        | <0,5 mg/ml<br>One CENP-B spot = 0,5µl on each strip     | >80%  |
| PM-Scl 100 antigen                       | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips                        | <0,5 mg/ml<br>One PM-Scl 100 spot = 0,5µl on each strip | >80%  |
| PM-Scl 75 antigen                        | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips                        | <0,5 mg/ml<br>One PM-Scl 75 spot = 0,5µl on each strip  | >80%  |

|                                   |   |                                      |   |          |
|-----------------------------------|---|--------------------------------------|---|----------|
| Ku antigen                        | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One Ku spot = 0,5µl on each strip                 | >80%     |
| RNA Polymerase III antigen        | Recombinant, human, expressed in E.coli                         | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One RNA Polymerase III spot = 0,5µl on each strip | >80%     |
| RNP 68kD/A/C antigen              | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One RNP 68kD/A/C spot = 0,5µl on each strip       | >80%     |
| Th/To antigen                     | Recombinant, human, expressed in E.coli                         | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One Th/To spot = 0,5µl on each strip              | >80%     |
| Fibrillarin antigen               | Recombinant, human, expressed in E.coli                         | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One Fibrillarin spot = 0,5µl on each strip        | >80%     |
| NOR-90 antigen                    | Recombinant, human, expressed in E.coli                         | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One NOR-90 spot = 0,5µl on each strip             | >80%     |
| SSA/Ro 52kD antigen               | recombinant, human, expressed in Baculovirus-infected Sf9 cells | Biomarker (antigen) coated on strips | <0,5 mg/ml<br>One SSA/Ro 52kD spot = 0,5µl on each strip        | >80%     |
| Protein L                         | Bacterial (from Peptostreptococcus magnus)                      | Reactive (positive) control          | 0,01 mg/ml<br>One RC spot = 0,5µl on each strip                 | >95%     |
| Streptavidin-Alkaline Phosphatase | Bacterial (from Streptomyces avidinii)                          | Cut-off (negative) control           | < 0,1 µg/ml<br>One CO spot = 0,5µl on each strip                | Unknown. |
| NBT-BCIP                          | Synthetic (chemical substance)                                  | Substrate for alkaline phosphatase   | 0,2 mg/ml   | ≥ 98%    |

#### 4. MATERIAL REQUIRED BUT NOT PROVIDED

##### BDI I:



##### The BlueScan and Dr Dot Software:



The BDI I is an instrument that performs the various steps of incubation and washing of D-tek's immunodot strips, from the deposit of the sample to the final colour development. The maximum capacity is of 24 strips which are incubated simultaneously. Each strip is associated with a cartridge containing the various reagents making it possible to carry out the test. The BlueDiver Instrument has a barcode reader which controls the correct association between a strip and its cartridge.

Prior training is strongly recommended (see your distributor).

Please consult The User Manual before using the BDI I.

The BlueScan and Dr Dot Software are intended for reading test results of D-tek immunodot strips. The Dr Dot Software and BlueScan scanner must be used in combination.

The scanner has been specifically developed for the reading of the strips with "BlueDiver" design. Based on the image of the scanned strips, the Dr Dot Software converts the intensity of each dot/line into a numerical value (the numerical scale is based on a greyscale). Results are expressed in arbitrary units (from 0-100). 1-24 strips can be read. Prior training is strongly recommended (see your distributor).

Please contact your distributor to obtain the latest version of Dr Dot Software.

Please consult The User Manual before using the BlueScan and Dr Dot Software.

##### BDI II:



The BlueDiver Instrument II is an instrument that performs the various steps of samples pipetting, incubation, washing, drying and reading of D-tek's immunodot strips, from the deposit of the sample tube to the final reading of the strips.

The maximum capacity of the BDI II is of 24 strips which are incubated simultaneously. Each strip is associated with a cartridge containing the various reagents making it possible to carry out the test. The BDI II has a barcode reader which controls the correct association between a strip and its cartridge.

It includes the BlueScan and Dr Dot reading system.

Prior training is mandatory (see your distributor).

Please consult The User Manual before using the BDI II

**Other Material:** Micropipettes, absorbent paper, protective equipment (see point 6).

#### 5. STORAGE

The test kit must be stored at a temperature between +2°C to +8°C throughout its validity period (see expiration date on the kit). Do not freeze.

After initial opening of the kit, unused reagent cartridges must be stored at 2-8°C protected from (sun)light preferably inside the original kit box.

Unused strips must be placed back into the provided pouches, sealed and stored at 2-8°C preferably inside the original kit box.

When stored properly, all test kit components are stable until the indicated expiry date.

## 6. SAFETY PRECAUTIONS

- All reagents are for in vitro diagnostic and professional use only. The test kit should be processed by trained technical staff only.
- The reagents in the kit are considered as not dangerous, as the concentrations of potentially dangerous chemicals are below the thresholds specified by European regulations:

| Name              | CAS       | EINECS | Concentration in strip | Classification according to Regulation EC 1272/2008<br>Significance<br>H Phrases |
|-------------------|-----------|--------|------------------------|--|
| Cellulose Nitrate | 9004-70-0 | -      | < 5 %                  | Flam. Sol. 1<br>H228   |

Annex VI to Regulation (EC) No 1272/2008: Index N°: 603-037-00-6; Commission Regulation (EU) 2015/830; 3.2.1

| Name | CAS        | EINECS | Concentration in mixture | Classification (in concentrated form) according to Regulation EC 1272/2008<br>Significance<br>H Phrases  |
|------|------------|--------|--------------------------|--|
| MIT: | 55965-84-9 | -      | < 0,0015 %               | Acute Tox. 2 H330<br>Acute Tox. 2 H310<br>Acute Tox. 3 H301<br>Skin Corr. 1 C H314; C ≥ 0,6%<br>Eye Dam. 1 H318; C ≥ 0,6%<br>Skin Sens. 1 A H317 ; C ≥ 0,0015%<br>A Aquatic Acute 1 H400<br>Aquatic Chronic 1 H410 |

Annex to Commission Regulation (EU) 2018/1480; Index Number: 613-167-00-5 ; Commission Regulation (EU) 2015/830; 3.2.1

| Name             | CAS        | EINECS    | Concentration in mixture | Classification (in concentrated form) according to Regulation EC 1272/2008<br>Significance<br>H Phrases                    |
|------------------|------------|-----------|--------------------------|--|
| NaN <sub>3</sub> | 26628-22-8 | 247-852-1 | < 0.1 %                  | Acute tox. 2<br>H300<br>Acute tox. 1<br>H310<br>STOT RE 2<br>H373<br>Aquatic acute 1<br>H400<br>Aquatic chronic, 1<br>H410 |

Annex VI to Regulation (EC) No 1272/2008: Index Number: 011-004-00-7; Commission Regulation (EU) 2015/830; 3.2.1

| Name | CAS      | EINECS    | Concentration in mixture | Classification (in concentrated form) according to Regulation EC 1272/2008<br>Significance<br>H Phrases |
|------|----------|-----------|--------------------------|---|
| NBT  | 298-83-9 | 206-067-4 | < 0,01%                  | Acute tox. 4<br>H302  |

Nevertheless, these chemicals are toxic in concentrated form. Therefore, contact with the skin, eyes or mucous membranes should be avoided by using suitable individual protection (gloves, laboratory coat, goggles). As with any chemical containing specific hazards, the product/components of the product should only be handled by qualified personnel and with the necessary precautions.

- Patient samples should be handled as if they could transmit infectious diseases; they therefore require suitable protection (gloves, laboratory coat, goggles). In any case, GLP should be applied with all the general or individual safety rules in force.
- Waste disposal: Patient samples, incubated test strips and used cartridges should be handled as infectious waste. The boxes and other containers do not need to be collected separately, unless stated otherwise in official regulations.
- The device is containing substances from animal, human, and bacterial origins (cfr 3.3) at very low concentration. All these substances were selected in order to not contain any microbial or transmissible agents and are non-toxic at the concentration used in the device. Nevertheless, laboratory good practice at user site (glasses, gloves) is necessary.

## 7. RECOMMENDATIONS

- D-tek and its authorized distributors cannot be held responsible for damages caused indirectly or due to a change or modification in the indicated procedure, an improper use of the kit and / or the use of an incomplete or damaged kit. The use of this kit is reserved for qualified technical personnel only.
- D-tek's responsibility is limited in all cases to the replacement of the kit.
- In the event of a serious incident (injury, deterioration in health, or death) with this IVD device, please report it immediately to the manufacturer (see address below) and to the competent authority in your country.

## 8. SAMPLE COLLECTION, HANDLING AND STORAGE

Sera with particles should be centrifuged at low speed. Blood samples should be collected in dry tubes. Please avoid using a pool of different sera, as this can lead to inconsistent results (see point 10.4). After separation, the serum samples should be used immediately or aliquoted and stored at 2-8 °C (for maximum 14 days) or frozen at -20°C (for longer storage periods, maximum 13 months). Repeated freezing/ thawing cycles of the samples must be at a maximum of 10 cycles.

## 9. ASSAY PROCEDURE


### BASIC INFORMATION, HANDLING AND TIPS:

#### TEST PROCESS principle:

After the manual loading of the strips and reagent cartridges, the incubation and washing steps are automatically processed by the BDI; the continuous up and down agitation of the strips in the wells of the ready-to-use reagent cartridges ensures an efficient circulation of the fluids over the strips. The whole test procedure is run at room temperature.

#### Description of the STRIPS:

**The reactive (front) side** of the strips is coated with antigens which appear as faint blue dots. This coloration ensures that all antigens have been correctly spotted onto the membrane. The coloration disappears during the processing of the test. This front side also displays a strip number and a 2-dimensional square barcode for traceability of the strips after removal from the BDI at the end of the test.

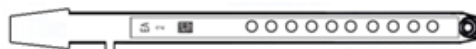

**The non-reactive (back) side** of the strips displays both alphanumeric and bar-coded information for identification of the strip type and lot number by the BDI.



The strips must be manually inserted into the dedicated clamp before starting the automated process (see 9.1 and 9.2 *Test Preparation* hereafter). During this operation, avoid touching the membrane zone of the strips with fingers. Always wear laboratory gloves and use the plastic parts (strip support) for handling or manipulation.

#### Description of the CARTRIDGES: (see image page 1)

The reagent cartridges are composed of 7 different wells filled with ready-to-use reagents. The cartridges are sealed, and the reagent wells are hermetically separated. The sealing must be removed before starting the test. Once opened, manipulate the cartridges with care in order to avoid reagent spilling and contamination from well to well.

The rear (back) side of the cartridges is labelled with both alphanumeric and bar-coded information for identification of the cartridge type and lot number by the BDI.

The cartridges must be manually loaded onto the dedicated cartridge holder before starting the automated process ((see 9.1 and 9.2 *Test Preparation* hereafter). The front and rear (back) sides of the cartridges have, respectively, a bottom triangular and two (bottom + top) square plastic edges for secure position and orientation into the holder.

#### Description of the CONTROLS:

The **Positive Control or RC (Reaction Control)** consists of a protein (protein L) fixing all the immunoglobulins present in the test sample. If the test has been carried out correctly, this control will show a colouring at the end of the test (with an intensity depending on the effective concentration of immunoglobulins in the sample).

The absence of any colouring of this dot at the end of the test may indicate that the sample has not been pipetted on the strip (see 10.4 *Troubleshooting*).

The **Negative Control or CO (Cut-Off Control)** consists of a protein (streptavidin – alkaline phosphatase) reacting with the enzymatic substrate and with certain constituent elements of the tested sample. If the test has been carried out correctly, this control is coloured at the end of the test, with a signal depending on the kinetics of the substrate and the characteristics of the sample. The intensity of this control serves as a threshold value for the final interpretation of the results (see 10 *INTERPRETATION OF RESULTS*).

#### STRIPS/CARTRIDGES association

The strips and cartridges of a same test kit share the same lot number and are dedicated to being associated in lot-specific pairs. Do not associate a strip and a cartridge with different lot numbers as this will be detected as an invalid setting by the BDI and will stop the process.

As long as each strip/cartridge pair is valid, the BDI can process strips/cartridges associations of different kits; However, only kits having the same protocol number (same incubation time and sequence) can be processed together in one same run (please refer to the protocol number indicated under the kit reference at the top of first page).

### 9.1 Test preparation on BDI I

Before any usage of the BDI, please refer to the manual of use supplied with the instrument.

- Allow all kit components to equilibrate at room temperature (+18°C to +25°C) before use.
- A working list (either edited from Dr Dot Software, or external) should always be prepared for easy loading and correct association of strips, cartridges and patient samples.
- Make sure that the cartridge holder is fixed in its emplacement in the BDI.
- Make sure that the BDI is plugged in.

The following steps sequence summarizes the loading and preparation of the BDI, test strips, reagent cartridges and patient samples before starting the test. For detailed information or in case of any problem met at one of the following steps, please refer to the Manual of Use of the BDI.

1. Switch ON the BDI and wait a few seconds until the date and time are displayed on the touch screen.
2. Confirm the correct Date and Time by pressing **V** on the touch screen (in case of first use or for reset, refer to the manual of use of the BDI) → "**Initialize?**" is displayed on the screen.
3. Confirm Initialization by pressing **V** on the touch screen → the horizontal arm of the instrument automatically moves forward to a central (stand-by) position → "**Load strips (24)**" is displayed on the screen.
4. (Do not set nor confirm the number of strips at this step). Remove the clamp from its emplacement on the arm by gently pulling it upwards and load the strips to be tested: handle the clamp with numbered side facing up (open position) and insert the strips, also with numbered (reactive) side facing up, by slipping the upper plastic part (tongue) into the dedicated holes of the clamp. Apply a gentle pressure to ensure that the plastic tongue has reached the bottom end of the hole.

Notes:



- Always start loading into position 1 of the clamp (left side) and do not leave empty spaces between the strips!  
After complete loading, check visually the vertical, horizontal and lateral alignment of the strips. Any obvious misalignment should be corrected by unloading the strip(s) from the clamp and loading them again.  
Be careful: any plastic bits remaining after breaking apart the individual strip holders may hinder the processing on the instrument and/or the reading with the BlueScan scanner; please remove them with scissors.
5. Replace the clamp in its emplacement on the arm by gently pushing it downwards.
  6. Set the number of loaded strips using the up and down arrows on the touch screen.
  7. Confirm the number of loaded strips by pressing **V** on the touch screen → the horizontal arm automatically moves backward to stand over the alignment holes of the cartridge holder → **"Check alignment"** is displayed on the screen.
  8. Use the "JOG" function on the screen to check the correct alignment of the strips: maintain a gentle pressure on the down arrow on the touch screen until the bottom of the strips enters the alignment holes of the cartridge holder. If correctly aligned, the strips will not touch the outlines of the holes.  
**Note:**  
In case of misalignment (contact of the strips with the cartridge holder), please refer to the Manual of Use of the BDI).
  9. Confirm the correct alignment of the strips by pressing **V** on the touch screen → the BDI lowers the strips completely into the alignment holes and reads the barcodes of the strips → after complete barcode reading, **"Load reagent"** is displayed on the touch screen.  
**Note:**  
In case of failure to read one or more strip barcode(s) (flashing LED at the unread position), please refer to the Manual of Use of the BDI.
  10. Unseal the reagent cartridges and insert them under their respective strips in the dedicated notches of the cartridge holder.
  11. Confirm complete loading by pressing **V** on the touch screen → the BDI reads the barcodes of the cartridges and checks the correct association with the strips → after complete barcode reading, the number of strips (validated strips/cartridges associations) is displayed on the screen.  
**Note:**  
In case of failure to read one or more cartridge barcode(s), or in case of detection of a wrong strip/cartridge association (flashing LED at the corresponding position), please refer to the Manual of Use of the BDI.
  12. Confirm the number of strips by pressing **V** on the touch screen → the protocol number identified on the barcodes is displayed on the screen (**Protocol ID xx**).
  13. Confirm the protocol number by pressing **V** on the touch screen → **"Please close cover."** is displayed on the screen.
  14. Close the cover of the BDI and confirm closing by pressing **V** on the touch screen → the BDI proceeds to a first washing (pre-treatment) step by incubating the strips into the 2<sup>nd</sup> well of the cartridges (processing time: 1 minute) → At the end of the wetting step, **"Please open cover."** is displayed on the screen.
  15. Open the cover of the BDI and confirm opening by pressing **V** on the touch screen → the horizontal arm automatically moves forward to the front of the instrument and swings the strips to an oblique position → **"Dry strips"** is displayed on the screen.
  16. Dry the strips by gently applying absorbent paper onto the basis of the bottom small cavity (sample loading hole).
  17. Confirm drying by pressing **V** on the touch screen → **"Apply samples"** is displayed on the screen.
  18. Apply samples by pipetting 10 µl of patient serum into the bottom sample loading holes of the strips.  
**Note:**  
If preferred the 10µl of the serum can be directly pipetted into the Diluent Buffer ("Well I") of the cartridge. This operation can be done at any time from opening of the cartridges (see 9.1.10).
  19. Confirm samples' loading by pressing **V** on the touch screen → **"Please close cover"** is displayed on the screen.  
Close the cover of the BDI and confirm closing by pressing **V** on the touch screen → the BDI starts the test automatically by proceeding the steps sequence of the protocol (see 9.3). After completion of the process, the clamp moves to a central (stand-by) position in the BDI to allow easy manipulation of the clamp. The instrument beeps and **"Finished test"** is displayed on the screen.
  20. Gently apply absorbent paper onto the basis of the strips to remove liquid from the bottom small cavity (sample loading hole) and allow the strips to dry for 30 minutes before interpretation of the results. The interpretation must be done in the 24 hours following the test processing. In case of use of the BlueScan for help of results interpretation, please leave the processed strips attached to the clamp

#### TEST DATA REGISTRATION

The test protocol can be downloaded by pressing the USB stick symbol and following the indications on the screen (Insert USB → Writing USB → Remove USB). This step is not obligatory but is highly recommended for traceability and regulatory matters.

#### 9.2 Test preparation on BDI II

Before any usage of the BDI II, please refer to the manual of use supplied with the instrument.

- Allow all kit components to equilibrate at room temperature (+18°C to +25°C) before use.
- All the preparatory steps requiring the operator's intervention are clearly indicated in the BDI's user interface. It is the instrument which indicates the number and type of tests to be run according to the indications made by the operator at the sample identification stage.  
The operator is guided by the user interface from the insertion of the samples and kits to be tested onwards, to the final interpretation of the results.
- Remember to open the reagent cartridges before inserting them into the holder.

#### 9.3 Test Processing (Protocol 02 for all D-tek immunodot kits on BDI I and BDI II):

| Step | Description   | Processing time |
|------|---|-----------------|
| 01.  | The strips are incubated into the 1 <sup>st</sup> well of the cartridge ( <i>Diluent Buffer</i> ). Upon contact with the liquid in the wells and agitation, the pre-loaded patients' samples (see 9.1.18) are released from the small cavity at the bottom of the strips and are diluted in the buffer. | 30 min          |
| 02.  | The clamp moves forwards and the strips are incubated into the 2 <sup>nd</sup> well of the cartridge ( <i>Wash Buffer</i> )   | 2 min           |
| 03.  | The clamp moves forwards and the strips are incubated into the 3 <sup>rd</sup> well of the cartridge ( <i>Wash Buffer</i> )   | 2 min           |
| 04.  | The clamp moves forwards and the strips are incubated into the 6 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )   | 2 min           |
| 05.  | The clamp moves backwards and the strips are incubated into the 5 <sup>th</sup> well of the cartridge ( <i>Conjugate</i> )  | 10 min          |
| 06.  | The clamp moves backwards and the strips are incubated into the 4 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )  | 2 min           |
| 07.  | The clamp moves backwards and the strips are incubated into the 3 <sup>rd</sup> well of the cartridge ( <i>Wash Buffer</i> )  | 2 min           |
| 08.  | The clamp moves backwards and the strips are incubated into the 2 <sup>nd</sup> well of the cartridge ( <i>Wash Buffer</i> )  | 2 min           |
| 09.  | The clamp moves forwards and the strips are incubated into the 7 <sup>th</sup> well of the cartridge ( <i>Substrate</i> )   | 10 min          |
| 10.  | The clamp moves backwards and the strips are incubated into the 6 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )  | 2 min           |

## 10. INTERPRETATION OF RESULTS

For **BDI I**: a visual (qualitative) interpretation of the results is possible, however the use of the BlueScan scanner and the Dr Dot software is generally recommended for more precision and for a semi-quantitative interpretation.

For **BDI II**: the semi-quantitative interpretation of the results is systematically performed at the end of the test.

**IMPORTANT NOTICE: The positivity of all parameters of this kit is NOT possible and in such a case the test is not valid. An additional test must be performed to establish the diagnosis.**

### 10.1. Qualitative Interpretation

1. Remove the clamp from the BDI and unload the strips from the clamp.
2. Place the strips with reactive face up onto the marked fields of the Diver Interpretation Template provided with the kit. This will indicate the respective positions of the different controls and antigens on the membrane.
3. The first upper dot (Positive Control) must be positive for all patients.  
*Only a clearly coloured Positive Control Dot ensures your results are valid and operation was correct and/or kit components were not degraded. If the first upper dot is not coloured, the test has failed and cannot be interpreted further.*
4. Compare the **specific antigen dots** to the **Negative Control Dot** (which always is the last bottom dot). The colour intensity of the Antigen dots is directly proportional to the titer of the specific antibody in the patient sample.  
*The colour intensity of the Negative Control Dot may vary depending on the sample characteristics. If the sample is free of interfering substances the Negative Control Dot may be even close to uncoloured. In contrast, a highly coloured Negative Control Dot indicates a high rate of unspecific binding in the sample.*

#### **POSITIVE RESULT:**

A sample is positive for a specific antibody if the colour intensity of the corresponding Antigen dot is higher than the intensity of the Negative Control Dot.

#### **NEGATIVE RESULT:**

A sample is negative for a specific antibody if the colour intensity of corresponding Antigen dot is lower than or equal to the intensity of the Negative Control Dot.

NB: A weak coloration of an antigen dot, when close to the colour intensity of the Negative Control Dot may be difficult to differentiate by visual inspection only. In such cases, it is recommended to use Dr Dot Software and scanning system (see 10.2) and refer to the corresponding instructions for more accurate interpretation.

### 10.2 Results semi-quantification: use of Dr Dot Software and Scanning system

The BlueScan scanner is a specially designed system for the reading of D-tek immunodot strips. It allows precise and easy insertion of test strips.

The Dr Dot Software allows a semi-quantification of results. Based on the image obtained, each result will be quantified in grayscale value and compared to the reference scale integrated in the BlueScan Cover.

These grayscale intensities will be transformed and displayed in arbitrary units (AU, from 0 to 100) based on the intensities of the controls (RC and CO, see point 9) present on the strip, according to the following conversion formula:

$$\text{Result of antigen X (AU)} = \frac{\text{Grayscale intensity of antigen X} - \text{Grayscale intensity of CO}}{\text{Grayscale intensity of RC} - \text{Grayscale intensity of CO}} * 100$$

1. Remove the clamp from the BDI. Leave the processed strips attached to the clamp. Be careful: the strips must be completely dry before starting the scanning step!
2. Insert the clamp, the reactive side of the strips facing down, into the dedicated emplacement in the cover of the BlueScan scanner.
3. Start scanning the strips using the Dr Dot Software.
4. The software semi-quantifies the results, and the interpretation of the obtained values is as follows

| Dr Dot arbitrary unit (AU) | Interpretation |
|----------------------------|----------------|
| < 5                        | Negative       |
| 5 – 10                     | Equivocal      |
| >10                        | Positive       |

For detailed information about the BlueScan and Dr Dot Software please refer to the Manual of Use of your Dr Dot Software

### 10.3 Important recommendations for the interpretation of results

1. D-tek's kits constitute a diagnostic aid. In consequence, no diagnosis can be established solely on the basis of our kits. The results should always be interpreted by taking into account the clinical examination, the patient's history and the results obtained by other methods.  
No single technique can rule out the possibility of false positive or false negative results. With this in mind, an indirect immunofluorescence test should, as far as possible, be carried out prior to the use of an immunodot kit (immunofluorescence being recognized as a reference method in autoimmunity).
2. The intensity of a result is not necessarily related to the degree of intensity of the disease, but rather to the level of antibodies detected.
3. Low titers of autoantibodies may occur in healthy patients. For this reason, low positive results (close to the CO, between 5 and 10 DrDot AU), although valid, should be considered equivocal. In such cases, the retesting of the patient, preferably by using a new sample, is recommended. If the result remains equivocal on retesting, other diagnostic tests and/or clinical information should be used to help determine the autoimmune status of the patient.
4. For various reasons, and under certain conditions, the kit may show a defect in performance (see 10.4 *Troubleshooting*). In such cases, the results are not valid and cannot be interpreted. It is recommended to repeat the test. If the error persists, please contact your distributor.
5. The intensity of the results may decrease when the device is used at the end of its life. However, the performance of the kit is not affected (detection of positives and negatives) under normal conditions of use and storage.
6. Sequential sampling (at different dates) of an autoimmune patient can sometimes lead to different results from one sample to another. This difference can have several reasons: the patient's treatment, the evolution of the disease, or a seroconversion. In

the specific case of seroconversion, the result can be positive for an autoantibody in an early sampling of the patient, and become positive for another autoantibody in a later sampling of the same patient.

#### 10.4 Troubleshooting

| Problem  | Possible causes + Action   |
|--|--|
| Discrepancy of results as compared to a reference method       | <ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>- incorrect volume dispensed</li> <li>- Use of two different samples of the same patient (see point 10.3.6) or wrong sample handling/storage between tests</li> <li>- erroneous visual interpretation</li> <li>- erroneous DrDot reading</li> <li>→ repeat the test</li> </ul> </li> <li>- Material <ul style="list-style-type: none"> <li>- Interfering substance in the sample</li> <li>- Sample is a pool of different human sera</li> <li>→ repeat the test and confirm by other methods</li> </ul> </li> <li>- Method <ul style="list-style-type: none"> <li>- intrinsic performance of the kit (see 11.2 <i>Analytical sensitivity and specificity</i>)</li> <li>- expired kit</li> <li>- stability problem</li> </ul> </li> </ul> <p><b>Please contact your distributor for any further technical support requests.</b></p> |
| Different results in the same batch or between several batches | <ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>- incorrect volume dispensed</li> <li>- erroneous visual interpretation or</li> <li>- bad DrDot reading</li> <li>→ repeat the test</li> </ul> </li> <li>- Method <ul style="list-style-type: none"> <li>- intrinsic performance of the kit (see 11.1 <i>Repeatability and Reproducibility</i>)</li> </ul> </li> </ul>  |
| Contamination between neighbouring strips                      | <ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>→ repeat the test</li> <li>- incorrect verticality of the strips on the BDI</li> <li>→ correct the verticality</li> </ul> </li> </ul> <p><b>Please contact your distributor for any further technical support requests.</b></p>  |
| RC absent or weak  | <ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- Serum not pipetted at all</li> <li>→ repeat the test</li> <li>- Patient with immunoglobulin deficiency</li> <li>→ repeat the test to confirm patient status</li> <li>- Damaged reagents</li> <li>→ check the integrity of the reagents</li> <li>→ contact your supplier if you suspect a problem</li> <li>- Spot not on the strip</li> <li>→ count the number of dots on the strip; if not correct, contact your supplier</li> </ul> </li> </ul>   |
| CO absent  | <ul style="list-style-type: none"> <li>- damaged reagents <ul style="list-style-type: none"> <li>→ check the integrity of the reagents, contact your dealer if you suspect a problem</li> </ul> </li> <li>- Spot absent from the strip <ul style="list-style-type: none"> <li>→ count the number of spots present on the strip, contact your distributor in case of incorrect number</li> </ul> </li> </ul>  |
| Non-specific bindings / high background / high CO value        | <p>Suspected presence of a contaminant or an interfering substance in the patient sample</p> <p>→ repeat the test and confirm through another method</p> <p><b>Please contact your distributor for any further technical support requests.</b></p>   |
| Barcode of the strips or cartridges cannot be read             | Manufacturing problem, please contact your distributor   |
| Kit content incorrect  | Manufacturing problem, please contact your distributor   |
| Positive results for all the biomarkers of the kit             | Problem with reagents, please contact your distributor   |

#### NOTE:

The major residual risks of the kit, as given in the risk analysis of the kit at the end of design (after mitigation), are the following:

- 1) Risk of false results based on a pipetting error (bad serum)
- 2) Risk of false results based on an interfering substance contained in the sample

#### 11. PERFORMANCES

##### 11.1 Repeatability and Reproducibility

Reference samples were tested for each antibody in successive statistically representative series, both in the same test as in different tests and between different batches in order to calculate the intra-assay, inter-assay and inter-lot variations respectively. In all the cases, the variations in colour intensity, when semi-quantified on the Dr Dot software, were within the following expected limits:

- CV ≤ 10% for intra-assay runs
- CV ≤ 15% for inter-assay runs
- CV ≤ 20% for inter-lot runs



## 11.2 Analytical sensitivity

Measurement range (semi-quantified results): From 0 AU (negative) to 100 AU (high positive).

Limit of detection: the lowest measured value of the test is 5 AU (considered as equivocal following the interpretation algorithm, see point 10.2)

As not any international standard is available for the autoantibodies, trueness of measurement and linearity are not applicable on this product.

## 11.3 Analytical specificity

- The main known interfering substances were tested on each biomarker of the present kit.  
For each concentration of interfering substance tested, the difference between the result of the sample without the interfering substance and the result obtained in the presence of the interfering substance did not exceed 15%.

| Interfering substance | Maximum Concentration | Intermediate Concentration | Minimum Concentration | Difference <15% |
|-----------------------|-----------------------|----------------------------|-----------------------|-----------------|
| Bilirubin             | 100 mg/dL             | 50 mg/dL                   | 25 mg/dL              | Yes             |
| Haemoglobin           | 200 mg/dL             | 100 mg/dL                  | 50 mg/dL              | Yes             |
| Cholesterol           | 224.3 mg/dL           | 112 mg/dL                  | 56 mg/dL              | Yes             |
| Rheumatoid factor IgM | ~500IU/ml             | ~300IU/ml                  | ~100IU/ml             | Yes             |

Note: It is impossible to test all the possible interfering substances described in the literature. Other interferences, amongst others drug-induced interferences, are possible.

- The high analytical specificity of the test is guaranteed by the quality of the antigen used. This kit detects IgG antibodies against Scl-70, CENP-A, CENP-B, PM-Scl 100, PM-Scl 75, Ku, RNA Polymerase III, RNP 68kD/A/C, Th/To, Fibrillarin, NOR-90 and SSA/Ro 52kD. No cross reactions with other autoantibodies have been found.

## 11.4 Clinical sensitivity and specificity

Sensitivity and Specificity were calculated from combined results obtained on clinically defined positive and negative EQAS controls and from historical data (external clinical evaluation on clinically defined positive and negative patients). These characterized samples (confirmed positive or negative for specific antibodies by reference laboratories and/or methodologies) were assayed following the test instructions. A detailed clinical report is available upon request.

| SENSITIVITY:  |                       |                        |                 |
|---|-----------------------|------------------------|-----------------|
| The percentage is established with the following calculation:<br>$\text{Sensitivity} = \frac{\text{True Positive Results}}{\text{True Positive Results} + \text{False Negative Results}}$ |                       |                        |                 |
| Antigen   | True Positive Results | False Negative Results | Sensitivity (%) |
| Scl-70  | 18                    | 0                      | >99             |
| CENP-A  | 24                    | 0                      | >99             |
| CENP-B  | 24                    | 0                      | >99             |
| PM-Scl 100  | 7                     | 0                      | >99             |
| PM-Scl 75   | 1                     | 0                      | >99             |
| Ku  | 3                     | 0                      | >99             |
| RNA Polymerase III  | 6                     | 1                      | 86              |
| RNP 68kD/A/C  | 40                    | 2                      | 95              |
| Th/To   | 5                     | 0                      | >99             |
| Fibrillarin   | 21                    | 0                      | >99             |
| NOR-90  | 8                     | 0                      | >99             |
| SSA/Ro 52kD   | 11                    | 0                      | >99             |

| SPECIFICITY:  |                       |                        |                 |
|---|-----------------------|------------------------|-----------------|
| The percentage is established with the following calculation:<br>$\text{Specificity} = \frac{\text{True Negative Results}}{\text{True Negative Results} + \text{False Positive Results}}$ |                       |                        |                 |
| Antigen   | True negative results | False positive results | Specificity (%) |
| Scl-70  | 272                   | 0                      | >99             |
| CENP-A  | 286                   | 3                      | 99              |
| CENP-B  | 205                   | 3                      | 99              |
| PM-Scl 100  | 68                    | 0                      | >99             |
| PM-Scl 75   | 38                    | 0                      | >99             |
| Ku  | 36                    | 0                      | >99             |
| RNA Polymerase III  | 56                    | 0                      | >99             |
| RNP 68kD/A/C  | 218                   | 5                      | 98              |
| Th/To   | 19                    | 0                      | >99             |
| Fibrillarin   | 62                    | 0                      | >99             |
| NOR-90  | 55                    | 0                      | >99             |
| SSA/Ro 52kD   | 51                    | 0                      | >99             |

Note: Sensitivity and specificity values of 100 % are strictly related to sample cohorts used in clinical evaluations. In theory, a diagnostic kit shouldn't be 100% sensitive or specific (at least > 99%).

## 11.5 Autoantibodies diagnostic values

|                         |   |
|-------------------------|---|
| Anti-Scl-70             | Diagnostic marker for Systemic Sclerosis (SSc)<br>Diagnostic specificity of 99%, sensibility of 10 % for limited SSc and up to 65% for diffuse SSc.   |
| Anti-CENP-A             | Diagnostic marker for Systemic Sclerosis (SSc).<br>Sensitivity of 57-82 % for patients with CREST syndrome (or other limited cutaneous forms of SSc) and in 3-12% of patients with diffuse cutaneous forms of SSc.<br>Detectable in 10-30 % of patients with Primary biliary cirrhosis (PBC).   |
| Anti-CENP-B             | Diagnostic marker for Systemic Sclerosis (SSc).<br>Sensitivity of 57-82 % for patients with CREST syndrome (or other limited cutaneous forms of SSc) and in 3-12% of patients with diffuse cutaneous forms of SSc.<br>Detectable in 10-30 % of patients with Primary biliary cirrhosis (PBC).   |
| Anti-PM-Scl 100         | Diagnostic marker for connective tissue diseases with myositis and symptoms of systemic sclerosis.<br>Diagnostic specificity of 50-70% for polymyositis/scleroderma overlap syndrome, of 20% for idiopathic myositis and of 10% for systemic Sclerosis (SSc).<br>Diagnostic sensitivity of 24-55% for polymyositis/scleroderma overlap syndrome, of 8-12% in patients with idiopathic myositis and of 1-16% for systemic Sclerosis (SSc). |
| Anti-PM-Scl 75          | More frequently found than PM-Scl 100.<br>May be detectable in patients without PM/Scl 100 antibodies (in 67% of cases of diffuse systemic Sclerosis).  |
| Anti-Ku                 | Found in 23% of patients with "primary" pulmonary hypertension<br>Found in 1.8 à 23% of patients with systemic lupus erythematosus (SLE).<br>Found in 1.2 à 14 % of patients with systemic sclerosis (SSc).<br>Found in 2 à 33% of patients with an overlap syndrome with myositis.   |
| Anti-RNA Polymerase III | Diagnostic specificity of 98-100% for Systemic Sclerosis (SSc)  |

|                   |   |
|-------------------|---|
| Anti-RNP 68kD/A/C | Diagnostic criterion of Mixed connective tissue disease (MCTD). Highly specific and extremely sensitive (100%) in the absence of Sm and dsDNA antibodies.<br>Found in 13 à 32 % of patients with systemic lupus erythematosus (SLE),<br>Found in 10 % of patients with systemic sclerosis (SSc)   |
| Anti-Th/To        | Marker of Systemic Sclerosis (SSc).<br>Detectable in 4-13% of SSc patients, more frequently found in limited cutaneous forms.<br>Also detectable in Raynaud's syndrome and idiopathic pulmonary fibrosis.   |
| Anti-Fibrillarin  | Diagnostic marker for Systemic Sclerosis (SSc), detectable in 4-22% of cases.<br>Prognostic marker for the development of isolated pulmonary hypertension.<br>Marker of xenobiotic exposure.  |
| Anti-NOR-90       | Since NOR-90 antibodies are very rarely detected it is difficult to make any evaluation of their potential clinical relevance. NOR-90 antibodies were originally considered to be commonly associated with systemic sclerosis (SSc), Raynaud's phenomenon and certain patients with Sjögren's syndrome. However more recent studies have indicated that they are not associated with any particular disease. They may be detected in diseases as varied as rheumatoid arthritis, systemic lupus erythematosus, drug-induced lupus, discoid lupus, celiac disease, sarcoidosis, alcoholic liver cirrhosis and hepatocellular carcinoma.  |
| Anti-SSA/Ro 52kD  | Found in various autoimmune diseases such as Systemic lupus erythematosus (SLE) (23%), Sjögren's Syndrome (SjS) (17-63%), Systemic Sclerosis (SSc) (20%), Rheumatoid Arthritis (RA) (8%), Primary biliary cirrhosis (PBC) (28%), autoimmune hepatitis (17%).<br>Often detectable in myositis patients with aminoacyl-tRNA synthetase, SRP antibodies, PM/Scl antibodies and Jo-1 antibodies<br>May be detectable in patients with systemic Sclerosis, in addition to Scl-70, CENP-B, CENP-A, RNA PIII and PM/Scl.<br>Severity marker of patients with Antisynthetase Syndrome and of patients with risk of pulmonary complications in systemic autoimmune rheumatic diseases (SARD) |

#### Publication references:

- 1: Mejia Otero C, Assassi S, Hudson M, Mayes MD, Estrada-Y-Martin R, Pedroza C, Mills TW, Walker J, Baron M, Stevens W, Proudman SM, Nikpour M, Mehra S, Wang M, Fritzler MJ; Canadian Scleroderma Research Group; Australian Scleroderma Cohort Study; Genetics versus Environment in Scleroderma Outcome Study. Antifibrillarin Antibodies Are Associated with Native North American Ethnicity and Poorer Survival in Systemic Sclerosis. *J Rheumatol.* 2017 Jun;44(6):799-805. doi: 10.3899/jrheum.160574. Epub 2017 Apr 1. PMID: 28365584; PMCID: PMC5457664.
- 2: Hudson M, Pope J, Mahler M, Tatibouet S, Steele R, Baron M; Canadian Scleroderma Research Group (CSRG), Fritzler MJ. Clinical significance of antibodies to Ro52/TRIM21 in systemic sclerosis. *Arthritis Res Ther.* 2012 Mar 6;14(2):R50. doi: 10.1186/ar3763. PMID: 22394602; PMCID: PMC3446416.
- 3: Mierau R, Moynzadeh P, Riemekasten G, Melchers I, Meurer M, Reichenberger F, Buslau M, Worm M, Blank N, Hein R, Müller-Ladner U, Kuhn A, Sunderkötter C, Juche A, Pfeiffer C, Fiehn C, Sticherling M, Lehmann P, Stadler R, Schulze-Lohoff E, Seitz C, Foeldvari I, Krieg T, Genth E, Hunzelmann N. Frequency of disease-associated and other nuclear autoantibodies in patients of the German Network for Systemic Scleroderma: correlation with characteristic clinical features. *Arthritis Res Ther.* 2011;13(5):R172. doi: 10.1186/ar3495. Epub 2011 Oct 21. PMID: 22018289; PMCID: PMC3308107.
- 4: Jeong S, Hwang H, Roh J, Shim JE, Kim J, Kim GT, Tag HS, Kim HS. Evaluation of an Automated Screening Assay, Compared to Indirect Immunofluorescence, an Extractable Nuclear Antigen Assay, and a Line Immunoassay in a Large Cohort of Asian Patients with Antinuclear Antibody-Associated Rheumatoid Diseases: A Multicenter Retrospective Study. *J Immunol Res.* 2018 May 2;2018:9094217. doi: 10.1155/2018/9094217. PMID: 29854849; PMCID: PMC5954951.
- 5: Ceribelli A, Krzyszczyk ME, Li Y, Ross SJ, Chan JY, Chan EK, Burlingame RW, Webb TT, Bubb MR, Sobel ES, Reeves WH, Satoh M. Atypical clinical presentation of a subset of patients with anti-RNA polymerase III--non-scleroderma cases associated with dominant RNA polymerase I reactivity and nucleolar staining. *Arthritis Res Ther.* 2011 Jul 22;13(4):R119. doi: 10.1186/ar3422. PMID: 21781293; PMCID: PMC3239357.
- 6: Karsten Conrad, Werner Schössler, Falk Hiepe, Marvin J. Fritzler, Book "Autoantibodies in systemic Autoimmune Diseases", Volume 2, third edition – 2015.

## 12. TEST LIMITATIONS

1. The results obtained with this confirmatory test are dependent on the intrinsic performance of the kit and must be considered as an aid to the final diagnosis, taking into account the results obtained by a reference technique and the clinical data of the patient.
2. In case of hyper-lipemic samples, it is recommended to centrifuge it before the pipetting of the 10µl of sample, which must be done into the supernatant.
3. Specific symptoms like hypomotility of the distal oesophagus, hypomotility of the small bowel, interstitial lung disease, primary pulmonary hypertension, systemic sclerosis-typical cardiac manifestation, or systemic sclerosis-typical renal manifestation can help to confirm the Systemic Sclerosis diagnosis.
4. The concentration of autoantibodies in a serum sample is not relative to the results provided by the device.
5. There is no link between the concentration of the different autoantibodies detected by the device, and the severity of its associated auto-immune diseases.



We Apply Science





We Apply Science

